

User Instruction for the Bariatric Toileting/ Showering Chair HMC106M

1. Introduction

The X399 is designed as a multi purpose showering and commode chair. The seat has a cut out for the ease of washing. This can be positioned either to the front of the chair or the rear. The chair should not be used for any other purpose other than its designed use – it is especially not for standing on.

Product specification	
Toilet seat height	585mm
Max width between arms	555mm
Toilet seat width	640mm
Toilet seat depth	460mm
Seat to top of back	390mm
Footprint	670mm x 980mm
Maximum user weight	254 KG (40 Stone)



Parts list	
Description	Part No
Oval Potty with Handle	71019F/H
Easy Clean PU Back	72PUSB
Easy Clean PU Seat	72PU500S
Slot in Plug	72PU500SP

2. Contents

- Bariatric Toileting/Showering Chair (X399)
- Oval Potty With Handle (71019F/H)
- Easy Clean PU Back (72PUSB)
- Easy Clean PU Seat (72PU500S)
- Slot in Plug (72PU500SP)
- User Instructions (this leaflet)

User Instruction for the Bariatric Toileting/ Showering Chair HMC106M

3. Assembly Instructions

- Carefully unpack and examine the product to ensure that it is complete.
- Before use, check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product.
- **DO NOT ATTEMPT TO USE IF PRODUCT IS DAMAGED.**

Push Handles (can be fitted in two ways):

- The handles can be turned inwards so that they run parallel with the chair back.
- Or the handles can be used facing out from the back of the chair at 90 degrees to the back.

Footrests:

- On each side of the front legs of the commode main frame there are two retaining lugs either side.
- Take the footrests and align the holes in the inverted “U” section with the lugs.
- Lower down into place.
- Rotate the foot rest towards the commode so that they clip into place.
- They can be rotated up and down as necessary, should they temporarily not be required.

4. Correct Use

- Please empty the potty at regular intervals and be aware of its maximum capacity.

Turning seat:

- The seat simply lifts off the frame and can either be forward or rearward positioned (relative to cut out)
- The seat aperture has a plug which can be fitted into the chair when access is not required.

Assistant Handles:

- They can be adjusted up and down by removing the pin clip and moving the handle to the desired height.
- The clip can then be refitted.

Castors:

- To lock: push down the lever on the castor body (all 4 are lockable).
- To unlock: push the lever in the opposite way.

Arms:

- To swing out of the way: lift the arm approximately 20-30mm and swing out
- To relocate lift, swing back in and drop the arm slot down onto the locating pin.

Health & Safety

- Customer/User Please Note: Do not attempt to move the product if the castors are locked.

71019F/H Potty:

- This product was developed for greater ease of use. The rear handle was designed intended to allow easy fitting and removal from the commode and greater control when carrying and emptying. The upper handle when put in the upright position locks into position for stability. To fold it down, push the handle vertically downwards to release the locking tabs before folding it backwards towards the rear handle.
- The potty can be fitted into the carrier (located underneath the seat) either from the front or the rear. This is possible when the seat is front or rearward facing.
- The upper handle will require to be folded down before fitting to the commode.

User Instruction for the Bariatric Toileting/ Showering Chair HMC106M



Handle folded down



Handle raised for carrying

5. Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this may damage the frame and nullify your warranty.
- It is recommended that the potty is thoroughly cleaned after every use, using an appropriate cleaning agent.

Use of power/jet washing & industrial washing machines:

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that frames are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the framework as it could cause permanent rusting of the frame (leave frames open in a warm environment for several hours to ensure all internal moisture evaporates).
- We cannot offer advice on industrial washing machines with regards to specific machine settings, as these will be dependant on the manufacturer's instructions.

6. Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

7. Warning

- This product is intended for users of a maximum of 250kg/40stone in weight.
- We recommend a full risk assessment should be carried out before use.
- It is advised that the frame is checked periodically for signs of wear and tear. Any problems please report to your supplier immediately.

8. Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage however we can advise that this product has been designed, manufactured and independently tested in the UK (including cyclic testing to represent real life usage).



User Instruction for the **Bariatric Toileting/** **Showering Chair HMC106M**

9. Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non hazardous and can be disposed of appropriately.

10. Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact: Harvest Healthcare Ltd
Sheaf House, Bradmarsh Way,
Bradmarsh Business Park,
Rotherham,
S60 1BW
Tel: +44 (0)1709 377172
E-mail: sales@harvesthealthcare.co.uk
