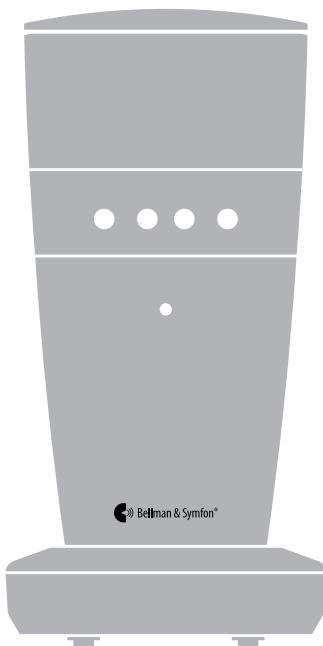
Visit Flash Receiver

Model BE1441/1442/1443

CZ

DE

DK

ES

FI

FR

GB

GR

HU

IT

NL

NO

PL

PT

SE

SRB

APP

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Bellman Visit Flash Receiver, BE1441/1442/1443

Thank you for choosing products from Bellman & Symfon.

The Bellman Visit System consists of a number of radio transmitters and receivers. The transmitters detect different events in the surrounding area and transmit a radio signal to the receivers. The receivers pick up this signal and provide indications using light, sound and/or vibration.

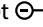
The transmitter determines what type of light, sound or vibration should be displayed so that the reason for the indication is evident.

Read through the entire user manual first and then start to install the system.

Refer to the illustration of the Bellman Visit System on the inside of the cover.

Getting started

Unpacking, installing and testing the unit

- 1 Connect the power supply unit to the socket . Press the Test Button (9). The Bellman Visit Flash Receiver will then start flashing and if a BE1270 Bellman Bed-shaker (accessory) is connected, it will vibrate.
- 2 A Bellman Visit transmitter is required to test the radio reception. Press the Test Button on the Bellman Visit Transmitter. The Bellman Visit Flash Receiver will then start flashing, emitting a light signal, and if a BE1270 Bellman Bed-shaker (accessory) is connected, it will vibrate.
- 3 It can be connected to an analogue telephone socket  with the BE9105 Telephone Flex (accessory) and an adapter plug (accessory). When someone rings the phone number, the Bellman Visit Flash Receiver will start flashing, the yellow LED  will light up and, if a BE 1270 Bellman Bed-shaker (accessory) is connected, it will vibrate.
- 4 Place the Bellman Visit Flash Receiver upright on a level surface or mount it on the wall using the BE9075 Bellman Wallmount Bracket (accessory). The receiver should be placed where it can best be seen.

Function

General

The BE1441/1442/1443 Bellman Visit Flash Receiver is a receiver within the Bellman Visit system for indoor use, which attracts the attention of the user with a flashing light, light signal, and also by vibration if a BE1270 Bellman Bed-shaker (accessory) is connected.

It is activated via radio signals from one of the Bellman Visit System transmitters or via direct connection to an analogue telephone socket.

The flash head can be rotated to point in the direction required. The flash head can, for example, be pointed towards a wall to prevent dazzle. A short press on the Test Button (9) activates BE1441/1442/1443 so that it repeats its last indication.

Radio key

On delivery all Bellman Visit units are tuned to the same Radio Key. If you have a neighbour with a similar system, you can change to different Radio Keys so that you do not affect each other's systems.

The Radio Key on this receiver can be changed by holding down the Test Button (9) for about five seconds until the LEDs  and  blink alternately. Then press the transmitter's test button so that the receiver's LEDs , , and  blink to confirm that the Radio Key has been changed. All units in a Bellman Visit System must have the same Radio Key in order to operate as a group. Refer to the user manual for the relevant unit.

Indicators and Signals

It is generally the transmitters in the Bellman Visit System that determine how the receivers will indicate an alarm. See the description in the relevant transmitter user manual for further information.

The Bellman Visit Flash Receiver has a function which allows you to easily check which alarm was the last one detected. Give a short press on the Test Button (9) and the last alarm will be repeated.

System indicators

The LEDs     that indicate which transmitter has activated the Bellman Visit Flash Receiver normally have the following meanings.

- Orange LED  indicates a baby cry transmitter.
- Green LED  indicates a door transmitter.
- Yellow LED  indicates a telephone transmitter.
- Red LED  indicates a fire alarm.

If the green  and yellow  LEDs blink alternately, this indicates that the Bellman Visit Flash Receiver is in radio key selection mode. The Bellman Visit Flash Receiver will then wait for a radio signal from a transmitter in the Bellman Visit System which will adjust the receiver to the same radio key as the transmitter is tuned to.

Flash light

When the Bellman Visit Flash Receiver is activated, the flash light flashes (2) with a sharp white light.

The flash head can be rotated to point in the direction required. The flash head can, for example, be pointed towards a wall to prevent dazzle.

Vibration

The Bellman Visit Flash Receiver can power a BE1270 Bellman Bed-shaker (accessory) connected to the socket . The bed shaker is placed under the pillow to wake the user up when the Bellman Visit Flash Receiver is activated.

Refer to the user manual for the relevant Bellman Visit transmitter for more information about vibration patterns.

Power supply

When the LED  lights up, the Bellman Visit Flash Receiver is correctly connected to the power supply.

Troubleshooting in brief

Problem	Solution
Nothing happens.	<ul style="list-style-type: none"> ▪ Check that the power supply unit is connected correctly. The LED  should light up green. ▪ Check that there is current in the wall socket.
The receiver is not activated.	<ul style="list-style-type: none"> ▪ Check the battery in the transmitters. ▪ Check that the receiver is not placed too far away from the transmitters by moving it closer to the transmitters. ▪ Check that the receiver is set to the correct radio key. For further information see Function/Radio key.
The receiver signals when no transmitter is activated.	<ul style="list-style-type: none"> ▪ Change the Radio Key on all units in the system. For further information see Function/Radio key.

- | | |
|--|---|
| 1 Rotatable reflector | 8 Telephone input for analogue telephone |
| 2 Flash | 9 Test button |
| 3 Orange LED | 10 Vibrator output |
| 4 Green LED | 11 Cable holder |
| 5 Yellow LED | 12 Table support |
| 6 Red LED | 13 Green LED |
| 7 Connector for power supply unit | |

For further information about the product in English, refer to the **Appendix**.

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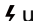
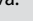
Settings

Please note: No adjustments are required for normal use. To change a settings, follow the steps below.

Changing the radio key

If your Visit system is activated for no reason, there is probably a nearby system that triggers yours. In order to avoid radio interference, you need to change the radio key on all units. The radio key switches are located on the transmitters.

Here is how you change the radio key:

- 1 Open the transmitter cover and move any radio key switch to the up = on position to change the radio key. See **Changing the radio key** for the relevant transmitter.
- 2 Press and hold the mute/test button on the top of the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 3 Press the test button/s on the transmitter within 30 seconds to send the new radio key.
- 4 All Visit LEDs on the receiver blink 5 times to show that the radio key has been changed. It then returns to normal mode.

Please note: All Visit units must be set to the same radio key in order to operate as a group.

Changing the signal pattern

When a transmitter is activated, the flash receiver lights up a LED, flashes and the bed shaker starts to vibrate with a certain pace. This is called signal pattern. The transmitters determine the pattern. Changing the transmitters' signal switch changes the signal pattern on the flash receiver.

The following signal patterns are available for the Visit system:
(Please note that the flash receiver does not emit any sound.)

868/315MHz Europe, Asia

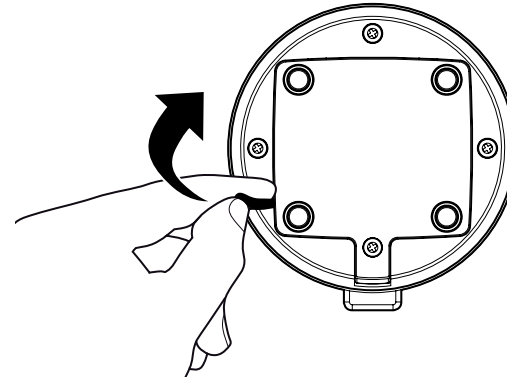
Type	LED pattern	Sound	Vibration	Flash
Green 1	Green is constantly lit	1 x ding dong, low-frequency tone	Separate	Yes
Green 2	Green blinks in sequences of two	2 x ding dong, low-frequency tone	Separate	Yes
Green 3	Green blinks in sequences of three	1 x ding dong, high-frequency tone	Separate	Yes
Green 4	Green blinks constantly	2 x ding dong, high-frequency tone	Separate	Yes
Yellow 1	Yellow is constantly lit	1 x ring, low-frequency tone	Short	Yes
Yellow 2	Yellow blinks in sequences of two	2 x ring ring, low-frequency tone	Short	Yes
Yellow 3	Yellow blinks in sequences of three	1 x ring, low-frequency tone	Short	Yes
Yellow 4	Yellow blinks constantly	2 x ring ring, high-frequency tone	Short	Yes
Orange 1	Orange is constantly lit	Baby	Rapid	Yes
Orange 2	Orange blinks in sequences of two	Baby	Rapid	Yes
Orange 3	Orange blinks in sequences of three	Baby	Rapid	Yes
Orange 4	Orange blinks constantly	Baby	Rapid	Yes
VMA	Red and Orange constantly blink alternately	VMA constant	Long	Yes
Fire alarm	Red blinks constantly	Fire alarm constant	Long	Yes

433MHz North America, Australia

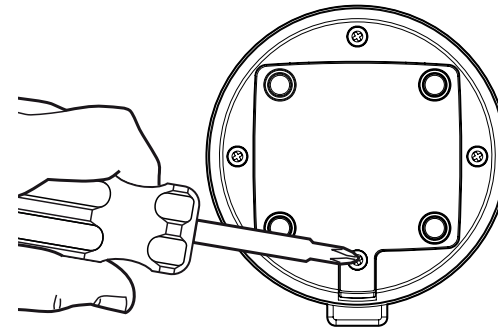
Type	LED pattern	Sound	Vibration	Flash
Green 1	Green is constantly lit	1 x Ding dong, Bass tone	Long	Yes
Green 2	Green is constantly lit	1 x Ring, Treble tone	Short	Yes
Green 3	Green is constantly lit	1 x Ding dong, Bass tone	Long	Yes
Yellow 1	Yellow is constantly lit	1 x Ding dong, Treble tone	Long	Yes
Yellow 2	Yellow is constantly lit	1 x Ring, Bass tone	Short	Yes
Yellow 3	Yellow is constantly lit	1 x Ding dong, Treble tone	Long	Yes
Orange 1	Orange is constantly lit	2 x Ding dong, Bass tone	Long	Yes
Orange 2	Orange is constantly lit	2 x Ring, Bass tone	Short	Yes
Red 1	Red is constantly lit	2 x Ding dong, Treble tone	Long	Yes
Red 2	Red is constantly lit	2 x Ring, Treble tone	Short	Yes
Red 3	Red is constantly lit	2 x Ding dong, Treble tone	Long	Yes
VMA	Red and Orange constantly blink alternately	VMA	Constant	Yes
Fire alarm	Red blinks constantly	Fire	Constant	Yes

Battery backup

BE1442

**Getting started**

- Remove the battery slip on the bottom of the unit before use.
- Please charge the unit for 24 hours to ensure full backup time.

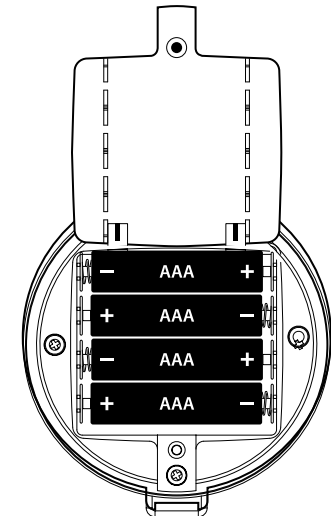
**Replacing batteries
(Done by the operator)**

- Remove the screw on the bottom of the unit to exchange the batteries.

CAUTION

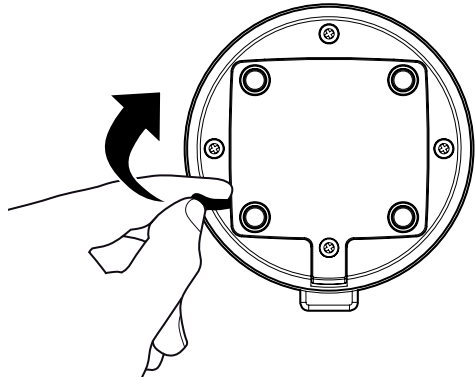
RISK OF EXPLOSION IF BATTERY IS REPLACED BY AN INCORRECT TYPE.

DISPOSE OF USED BATTERIES ACCORDING TO THE INSTRUCTIONS.

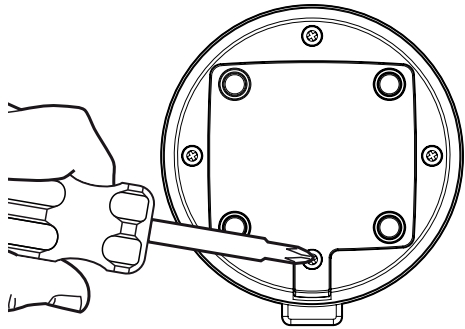
- Use only 4x1.2VNiMH 600AAA/900AAA Lixel rechargeable batteries to ensure full backup time.

Battery backup

Getting started

- Remove the battery slip on the bottom of the unit before use.
- Please charge the unit for 24 hours to ensure full backup time.

Replacing batteries (Done by the operator)

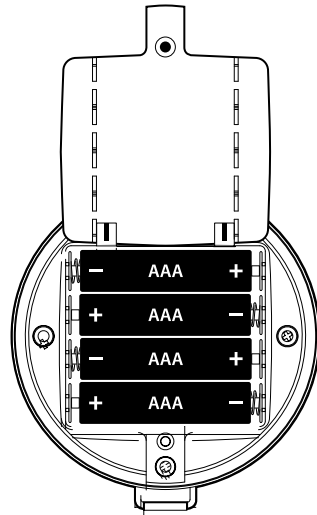
- Remove the screw on the bottom of the unit to exchange the batteries.

BE1443

CAUTION

RISK OF EXPLOSION IF BATTERY IS REPLACED BY AN INCORRECT TYPE.

DISPOSE OF USED BATTERIES ACCORDING TO THE INSTRUCTIONS.

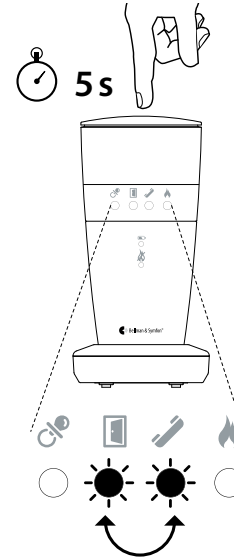
- Use only 4x1.2VNiMH 900AAA Lexel rechargeable batteries to ensure full backup time.

Smoke detector supervision

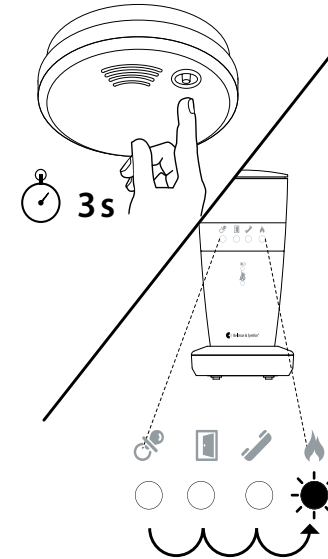
Pairing a smoke detector with the Flash receiver

1

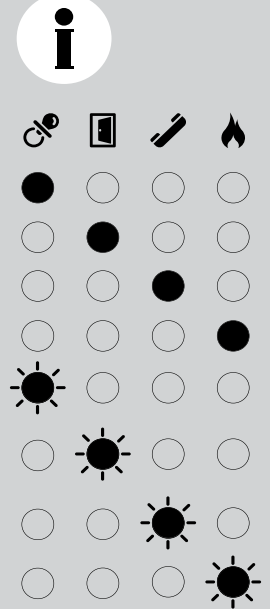
- Press (9) and hold the top button for 5 seconds.
- The flash receiver indicates that it is ready by switching between the green and the yellow LED light.
- Release the top button.

2

- Press and hold the test button on the smoke detector for more than 3 seconds.
- Release the button.
- The flash receiver will indicate that pairing was successful by switching between the orange, green, yellow and red LED lights three times.

BE1443

- This menu shows the position of the smoke detector.
- Attach the corresponding label (provided with each smoke detector) on the smoke detector unit so you can easily identify it later.

Flash receiver error identification

BE1443

If a paired smoke detector is faulty, the red LED  light on the Flash receiver will start to blink. By looking at the power LED  and the smoke detector LED  you can easily identify what is wrong. If you are using multiple smoke detectors, just enter the smoke detector menu to find the faulty unit.

Error	Indicaton	
	Power LED 	Smoke detector LED 
Low battery	<ul style="list-style-type: none"> RED, blinking 	<ul style="list-style-type: none"> AMBER
Lost connection	<ul style="list-style-type: none"> GREEN 	<ul style="list-style-type: none"> AMBER, blinking
Hardware failure	<ul style="list-style-type: none"> OFF 	<ul style="list-style-type: none"> AMBER, blinking

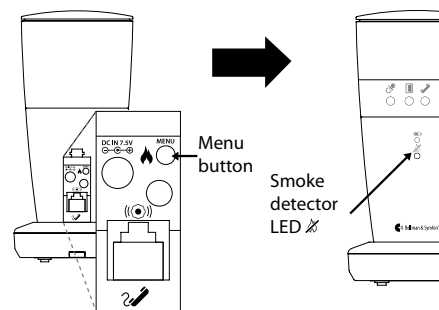
Trouble shooting

Most problems can be solved quickly by following the advice below.

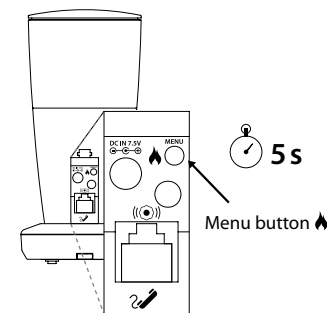
If	Try this
The smoke detector indicates low battery	<ul style="list-style-type: none"> Replace the battery using the correct batteries stated at the smoke alarm label.
The connection is lost	<ul style="list-style-type: none"> Check if the battery in the smoke detector is still OK. Move the units to their original position and verify that the error message disappears.
There is a hardware failure	<ul style="list-style-type: none"> Clean the smoke detector according to the smoke detector manual. If it is still faulty, replace the smoke detector.

Removing one smoke detector from the system

BE1443

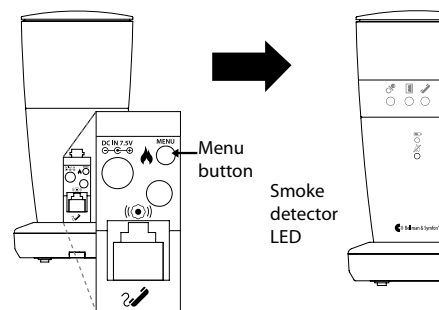
- Press the Smoke detector Menu button  on the backside of the unit.
- The Smoke detector LED  will blink green to indicate you are entering the menu system.
- Press the Smoke detector Menu button  as many times as needed to reach the smoke detector you would like to remove from the system.

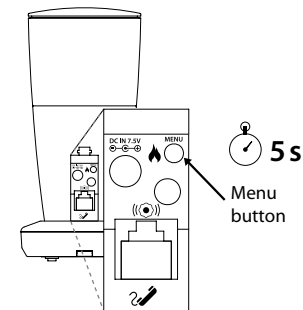
- Press and hold the Smoke detector Menu button  for more than 5 seconds.
- Release the button.
- The flash receiver will indicate the removal of the smoke detector by switching between the orange, green, yellow and red LED lights three times.

Removing all smoke detectors from the system

- Press the Smoke detector Menu button  on the backside of the unit.
- The Smoke detector LED  will blink green to indicate you are entering the menu system.

- To remove all smoke detectors in the system, press and hold the Smoke detector Menu button  for more than 5 seconds.
- Release the button.
- The flash receiver will indicate the removal of the smoke detector by switching between the orange, green, yellow and red LED lights three times.

Advanced programming

By using advanced programming, you can customize the signal pattern from a specific transmitter and event, displaying the LED colour and vibration pattern of your choice. The advanced programming overrides the radio key and pairs the units via the serial number. Please note that smoke alarms cannot be programmed for safety reasons.

Note: The transmitter must be activated as it is intended to be used in the system to generate the right signal.

This means that you can't always use the transmitter test button (see Default signal pattern for the relevant transmitter).

Here is how you program the receiver:

- 1 Press and hold the mute/test button on the receiver. The green and yellow Visit LEDs will start to blink alternately. While still holding down the button, activate the desired transmitter as intended. Release the button.
- 2 Scroll through the different **Visit LED options** by pressing the mute/test button on the receiver. Select the desired Visit LED colour by holding down the mute/test button until the power LED goes out and lights up again.
- 3 Scroll through the different **vibration options** by pressing the test button on the receiver (bed shaker required). Select the desired vibration pattern by holding down the mute/test button until the power LED goes out and lights up again.
- 4 The receiver will now show the new Visit LED colour and vibration pattern. Press the mute/test button briefly to end the demonstration. After a short while, it will return to normal mode.

Deleting the advanced programming

Follow the procedure below to delete the advanced programming.

- 1 Hold down the mute/test button on the receiver until the green and yellow Visit LEDs blink alternately. Release the button.
- 2 Press the mute/test button on the receiver 3 times in quick succession.
- 3 All Visit LEDs will light up for ~2 seconds to show that it has been deleted.

Testing

It is easy to test the flash receiver. If it does not work as described below, see the section **Troubleshooting**.

Testing the system

A Visit transmitter set to the same radio key as the flash receiver is required to test the system. You will need a bed shaker accessory to test the vibration.

- 1 Press the transmitter test button.
- 2 The flash receiver will:
 - Start to flash
 - Light up the corresponding Visit LED
 - If a bed shaker accessory is connected, it will start to vibrate

Testing the landline telephone connection

- 1 Connect the flash receiver to an analogue telephone socket via the telephone Input. You will need a telephone flex cable accessory and an adapter plug accessory.
- 2 Use for instance your mobile phone to call your landline telephone. The flash receiver will start to flashing and light up the yellow Visit LED. If a bed shaker accessory is connected, it will start to vibrate. To repeat the last indication, press the flash receiver test button.

Troubleshooting

Most problems with the flash receiver can be solved quickly by following the advice below.

If	Try this
Nothing happens.	<ul style="list-style-type: none"> Check that the power supply unit is connected correctly. The power LED should light up green. Check that there is current in the wall socket.
The receiver is not activated.	<ul style="list-style-type: none"> Check the battery in the transmitters. Check that the receiver is not placed too far away from the transmitters by moving it closer to the transmitters. Check that the receiver is set to the correct radio key. For further information see Radio key.
The receiver signals when no transmitter is activated.	<ul style="list-style-type: none"> Change the radio key on all units in the system. For further information see Radio key.

Technical information

Power supply

Mains power	7.5 V DC / 1500 mA Note: Only use the supplied power adapter. This product is also designed for IT power distribution system with phase to phase voltage of up to 230V. The socket-outlet shall be installed near the equipment and shall be easily accessible.
-------------	--

Battery backup	For BE1441: not applicable For BE1442: 4x1.2V Lexel 600/900mAh For BE1443: 4x1.2V Lexel 900mAh
----------------	--

Please dispose used batteries at municipal collecting points or dispense them to local stores free of charge.

Radio function

Power consumption	Active current: 900mA idle current: 10mA
Radio frequency	314.91/433.92/868.3 MHz depending on region

Number of Radio Keys	64 Radio Keys as standard. Special software can be used to increase these to 256 Radio Keys in increments of 64 per software purchase. Contact the nearest supplier for further information.
Coverage	The normal coverage between a transmitter and receiver in the Bellman Visit System is up to 250 metres (868MHz)/ 80m (433MHz)/ 50m (314.91MHz) with clear line of sight. Coverage is reduced if walls and large objects screen off the signals. Any thick walls constructed of reinforced concrete will greatly affect coverage. The system may also be affected by radio transmitters such as TV transmitters, computers, mobile phones, etc. This means that a unit may work perfectly in one part of the room but not at all in another.

Additional information

Programming pin (16)	For service personal only (Output: 3.3V/max. 150mA) Caution: Do not use, unit will be damaged when used.
----------------------	--

Activation

Radio	Visit system
Via analogue telephone nw. BE1441/BE1442	26 - 120 V RMS, 15 - 100 Hz. Via the Mobile phone detector, see the accessories section (Output: 3.3V/max 150mA)

Warning! The telephone port of the Flash receiver must be connected using 26 AWG or greater telecommunications wire.

Output signals

Output signals	Built-in flash light signal ~30 Candela Warning! Flashes can trigger epileptic seizures.
Vibrator power	2.0 – 4.0 VDC (max. 500mA)
Environmental requirements	Operating temperature: 0°C – 35°C Transport and storage temperature: -10°C – 50°C Relative humidity: 15% - 90%, non condensing
Dimensions Ø x H	BE1441: 70 x 140 mm BE1442/BE1443: 78 x 155 mm
Weight	308 g, 10.8 oz (three models)
Colour	White

Optional accessories

BE9075 Wall mount bracket
BE1270 Bed shaker
BE9105 Telephone flex cable
BE9250 Mobile phone sensor
Adapter plug for the appropriate country

FCC compliance statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Statement for Industry Canada

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to the following two conditions:

- (1) This device may not cause interference.
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

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Bellman Visit vilkkuvastaanotin voi aktivoida BE1270 Bellman tärhistimen (lisävaruste), joka on kytketty pistorasiaan (©). Täristin sijoitetaan tyynyn alle, niin että se herättää käyttäjän Bellman Visit vilkkuvastaanottimen aktivoituessa.