
# Instruction for use

IMM1006\_IFU.Version6.2018-11-01

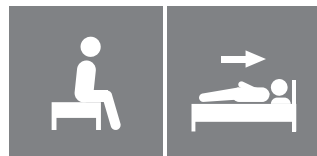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**immedia**  
a part of Etac


# Index

Illustrations .....	3
User manual – English .....	4
Bruksanvisning – Svenska.....	9
Bruksanvisning – Norsk .....	14
Brugervejledning – Dansk.....	19
Käyttöohjeet – Suomi.....	24
Gebrauchsanweisung – Deutsch .....	29
Gebruiksaanwijzing – Nederlands.....	34
Instruction d’utilisation – Français.....	39
Istruzioni per l’uso – Italiano .....	44
Instrucciones de uso – Español .....	49

See [www.etac.com](http://www.etac.com)

# Illustrations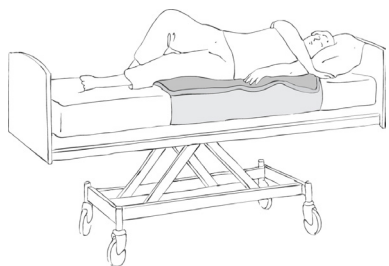

1

# General

## Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, read the manual carefully.

The person described as “the user” in this user manual is the person who is lying or sitting on the product. The carer or carers are the people who manoeuvre the product.

**This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the user or carer.**

The products comply with the standards applicable for Class 1 products in the European Council Directive MDD 93/42/EEC on medical devices.

At Etac we strive to improve our products all the time and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations

and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

For further information on the Etac transfer range, see [www.etac.com](http://www.etac.com).

The product can be scrapped in accordance with national regulations.

## Symbols in general

Washing

pH level

Read the user manual

User mass limit = maximum rated load

Do not bleach

Wipe off

Warning

The product can be scrapped according to national regulations

Tumbledry

Never leave on the floor

Production year, month and day

Manufacturer

Iron

Risk of sliding down

Batch no./Lot no.

Class 1 - European Council Directive MDD 93/42/EEC on medical devices

Do not dryclean

Keep out of the sun

## Intended use

4WayGlide - mini is used to turn self-supportive users, if necessary with carer assistance.

## Intended environment

Acute care, Long-term care, Home care.

# Practical handling

## Positioning

Position the nylon sheet centrally on the mattress, so that the “this side up” label faces the user.  
Secure the sheet under the mattress using the wings or position loosely on top of the mattress. Due to the non-slip surface on the back side, this is not displaced during turning/moving.

Position 4WayGlide - mini centrally on top of the sheet with the nylon side against the nylon sheet so that the “this side up” labels are facing each other.

## Use

### Positioning

Position the user on the mattress, using a ceiling or floor hoist. If a standing/sitting hoist is used or if the user is able to move him/herself to the edge of the bed, it may be beneficial to turn the mattress, so that the user is seated in the middle of the long side (140 cm) of the mattress. The user can now easily turn him/herself or can easily be turned in the bed, after which the mattress is in the correct position for further moving and turning.

### Turning

If the user is able to move him/herself, he/she can push him/herself away from the mattress and turn (Illustration 1). If assistance is needed, a carer grasps the functional handles under 4WayGlide - mini under the user's shoulders and hips, stands with one foot in front of the other and pulls the user over to the edge of the bed using weight transference. (If necessary, a second carer can push against the mattress.) Carer A bends slightly at the knee, and, with elbows close to the body, straightens up, thus turning the user on their side. Alternatively, the carer can pull the user towards him/herself: The carer moves to the opposite side of the bed, grasps the opposite functional handles and rolls the user towards him/herself, again using weight transference.

### Positioning

The positioning wedge is positioned at the user's back. The user is rolled or leans against the wedge until he/she finds a comfortable side position – other parts of the body can be positioned as required.

### Raising the bed headboard

When the bed headboard is raised, the user will tend to slide down a little until either the legs stop the downward movement or the carer places one or more pieces of loose non-slip material between the two nylon surfaces. If required, one carer can secure the excess material from 4WayGlide - mini under the mattress.

### Seated on the edge of the bed

Self-supportive users can rotate themselves using natural movement. Alternatively, a carer can lift the user's legs over the edge of the bed, possibly using a sling. The user is then raised to the seated position on the edge of the bed, by means of weight transference and the lever arm principle. The non-slip material along the edge of the mattress ensures that the user does not slide.

# Notice

## Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that carers receive instruction in manual handling. Etac offers advice and training for carers. For further information, contact Etac.

Always use the correct manual handling techniques

Encourage the user to assist where possible.

To ensure that the user feels safe and that every manual handling is smooth, always plan the manual handling in advance.

There is a risk that the user may slide off. Never leave the user alone on the edge of the bed.

Never leave the product on the floor.

Always conduct a risk assessment, and ensure that the assistive product can be used for the individual user and, in combination with other devices, that it is safe for the user and carers.

It is recommended that the guard rail is in place when the user is left on the glide system, unless a risk assessment has found that the user can be safely left on the glide system without the need of guard rails.

Always lock the glide system when you have completed the transfer.

Do not use the product as a sling.

If in any doubt - please contact Etac.

## Material and cleaning

### Material:

#### GlideMattress:

Top side: Polyester/cotton

Filling: Polyester fibre

Back side: Nylon

Handles: Polyester.

**NylonSheet:** Nylon, non-slip.

The product should be checked regularly, preferably each time it is used, and especially after washing.

Check that there is no damage to seams or fabric.

**Never try to repair a product yourself.**  
**Never use a defective product.**

### Washing instructions:

Recommended to wash the product before use.

Do not use fabric softener.

This product can be cleaned with a solvent-free cleaning agent with a pH level between 5 and 9, or a 70% disinfection solution.

## Products and accessories

---

Article no.	Product Description	Size (mm)
IM140/100	4WayGlide Mattress - mini	W1400xL1000
IM140/100NS	4WayGlide NylonSheet - mini	W1600xL1000

## Set

---

Article no.	Product Description	Size (mm)
IM4301	4WayGlide - mini set. Incl. IM140/100 + IM140/100NS	

## Combinations

---

Article no.	Product Description	Size (mm)
IM53NS	Positioning Wedge S	W300xL530xH200
IM53/80NS	Positioning Wedge L	W300xL800xH200

ILL.

GB

SV

NO

DA

FI

DE

NL

FR

IT

ES

# Service information

## Inspection

**Thank you for choosing an Etac product.** If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid.

**Warranty:** Two-year warranty on material and manufacturing defects, provided that the product is used correctly. Before taking a product into service always conduct a risk assessment according to national and/or local legislation.

### First inspection:

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description.
- Check that Short Instruction is enclosed – updated User Manual/Instruction for use can be downloaded from [www.etac.com](http://www.etac.com) or contact Etac Customer Service or your local distributor.

- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

### Periodic Inspection:

- Make sure that your Manual transfer system is always in perfect condition.
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.


### Visual inspection/Check the product:

Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.

### Stability test:

Try to bend the product and verify that it feels firm, stable and solid.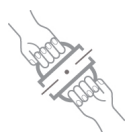

### Mechanical load or stress/Test handles:

Pull hard on the handles in opposite directions and check the material, seams and stitching.

### Test buckles:

Lock the buckle and pull in opposite directions, check the buckle and the stitching.

### Test low/high friction:

Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the product.  
Low friction – slides effortlessly  
High friction – no sliding/movement

## Check

### Manual transfer aids with handles, straps and buckles:

### Manual transfer aids for sitting and lying transfer:

### Manual transfer aids with high or low friction:

The product can be scrapped in accordance with national regulations

Read the User Manual

If in doubt, please contact Etac Customer Service or local distributor for more information and guidance.

For more information about training and the use of these products, contact the Etac transfer experts – [www.etac.com](http://www.etac.com).

Class 1 - European Council  
Directive MDD 93/42/EEC  
on medical devices

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Etac AB
Etac Immedia A/S