



Geberit AquaClean Mera Care Floor standing

Shower chair / equipment compatibility report

2nd edition

Simon Thomas

Updated Feb 2020

Index

- 3: Geberit AquaClean Care team contact details
- 4 – 6: Product overview
- 6 – 21: Shower chairs
- 22 – 29: Tilt in Space shower chairs
- 30 – 36: Paediatric equipment
- 37 – 42: Bariatric equipment
- 43 – 49: Toilet lifters
- 50 – 54: Miscellaneous Equipment
- 55 – 57: Product details & Manufacturer contact details
- 58: How to access shower chair mode



DEPEND ON INDEPENDENT EXPERTISE

OUR DEDICATED TEAM HAVE MANY YEARS OF
EXPERIENCE IN CARE AND ARE HERE TO HELP

SIMON THOMAS

SALES MANAGER (NATIONAL)

07738 404 822

simon.thomas@geberit.com

MAX BARRATT

NORTH WEST / YORKSHIRE & MIDLANDS

07715 758 827

max.barratt@geberit.com

DENIS HARLEY

SCOTLAND / CUMBRIA & THE NORTH EAST

07966 964 514

denis.harley@geberit.com

MATT SMITH

THE SOUTH EAST & LONDON

07738 404 828

matthew.smith@geberit.com

RICHARD WHEELER

THE SOUTH WEST & SOUTH WALES

07515 063 521

richard.wheeler@geberit.com



Revolutionary Features

Rimless ceramic – TurboFlush

App control

Wireless Opto remote

User remote control

Self servicing function

5 Year FREE Warranty

YouTube video support



Constant warm water

Industry leading **1 minute** Wash cycle

Whirlspray Shower Technology

Oscillating Wash

Adjustable wash arm positioning

5 programable user settings

Bluetooth Technology



Compatibility

With an estimated **50%** of wash dry toilets provided under a DFG grant being used with a type of shower chair or assistive equipment, compatibility is essential.

This report looks at the wide range of equipment available on the healthcare market that works satisfactorily with the Geberit AquaClean Mera Care wash dry toilet.



Shower Chairs

Attendant or self propel, there is a wide range of upright shower chairs available from several manufacturers.

Osprey Healthcare, Invacare, Freeway (Prism Medical), Aidservice, Chiltern Invadex, and Etac all have a range of upright shower chairs that are fully compatible with the Geberit AquaClean Mera Care shower toilet.





Attendant Sani Commode

Key Features

- Stainless Steel (Rust Resistant)
- Adjustable Seat Height 19"-22"
- Flexi Seat, Horseshoe Seat,
- Toilet Seat, Various apertures
- Detachable Foldaway Footrest
- Foldaway Padded Arms
- Compatible with Washer/Dryer toilets
- 30 Stone SWL



Self Propel Sani Commode

Key Features

- Stainless Steel (Rust Resistant)
- Adjustable Seat Height 19"-22"
- Flexi Seat, Horseshoe Seat
- Toilet Seat, Various apertures
- Detachable Foldaway Footrest
- Foldaway Padded Arms
- Compatible with Washer/Dryer toilets
- 30 Stone SWL



Bath Mobile Shower Toilet Chair

Key Features

- Easily dismantled
- Rust proof
- Flip away arms
- 130kg Safe working load
- Range of accessories



Freeway (Prism Medical)

Freeway T40 Auto

Key Features

- The Freeway T40 / T70 AUTO is for use with automatic toilets. Features pram handle backrest and seat with skirt as standard.

Dimensions

- The Freeway T40 Shower Chair range is available in three standard sizes, based on seat width:
 - 'A' - 540mm or 21"
 - 'B' - 490mm or 19"
 - 'C' - 440mm or 17"



Freeway (Prism Medical)

Freeway T70 Auto

Key Features

- The Freeway T40 / T70 AUTO is for use with automatic toilets. Features pram handle backrest and seat with skirt as standard.

Dimensions


- The Freeway T40 Shower Chair range is available in three standard sizes, based on seat width:
- 'A' - 540mm or 21"
- 'B' - 490mm or 19"
- 'C' - 440mm or 17"



The Raz AP GEB Shower Chair

Key features

- Attendant Propelled
- Stainless steel height adjustable frame
- 25 stone weight limit
- Seat depth – 400 or 480mm
- Overall length with footplate – 860mm
- 5 Year warranty
- Tension adjustable back rest

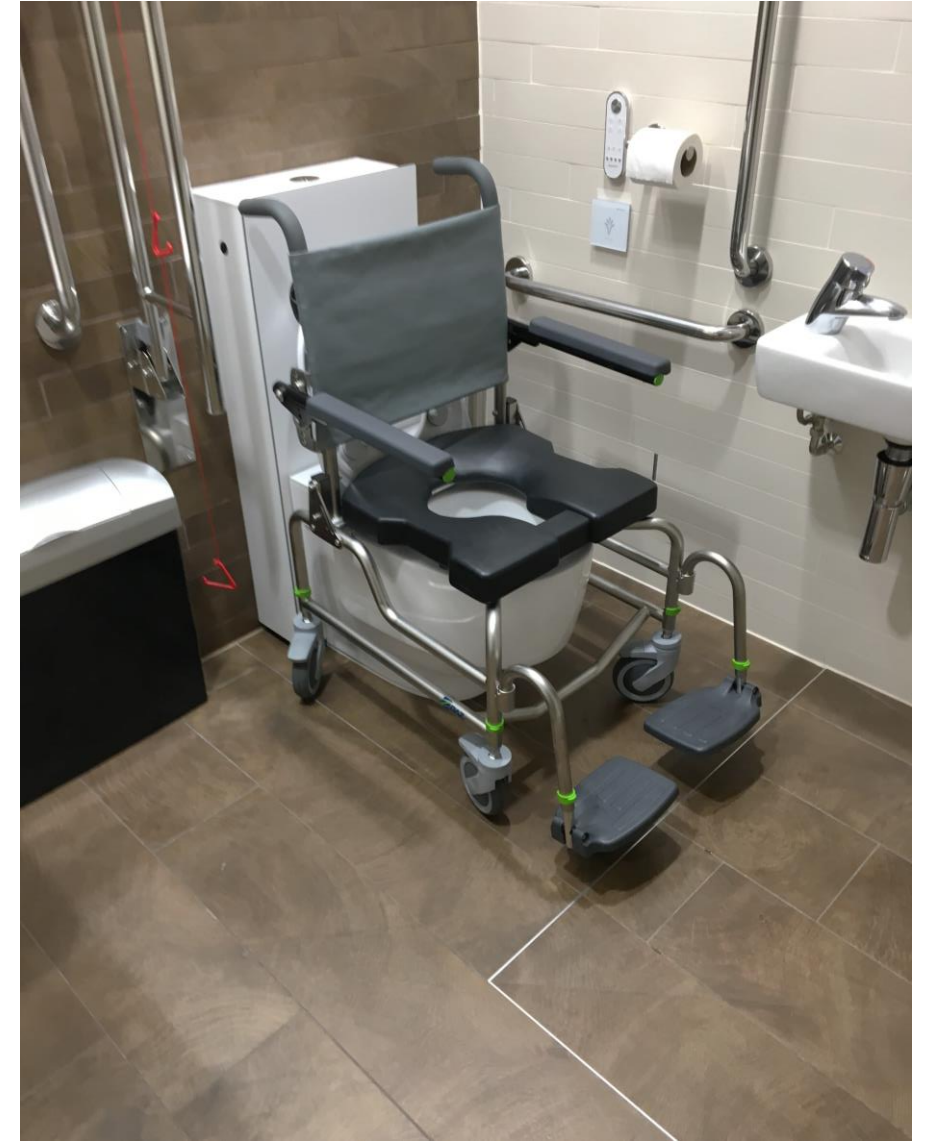

Freeway (Prism Medical)

Freeway T60A / T60B

Key Features

- Safe Working Load (SWL): 200kg (30 stone)
- Available as a shower, toilet or commode chair
- Perfect for use in tight spaces
- Anti-tip bars fitted to the rear as standard
- Pliable backrest moulds to the user while providing excellent support
- FREECOTE®, antibacterial and antimicrobial coating.
- **19" or 21" seat widths only as standard**



Invacare

Aquatec Ocean shower chair

Key Features

- Stainless steel frame
- Machine washable back rest
- Adjustable height seat settings
- Horse shoe Seat
- Fold away footplates
- Maximum user weight 180kgs



Flyta Active shower chair

Key features

- Stainless steel frame
- 17" or 19" Seat widths
- 15 Year warranty
- Toilet, Horseshoe, or full aperture Seat options available
- 24" Self propel wheels
- Webbed calf rest
- 18 stone user weight limit



The Raz SP GEB Shower Chair

Key features

- Attendant Propelled
- Stainless steel height adjustable frame
- 25 stone weight limit
- Seat depth – 400 or 480mm
- Overall length with footplate – 860mm
- 5 Year warranty
- Tension adjustable back rest



ETAC

Etac Clean Shower Commode Chair

Key Features

- Unique foot support
- Swing away arm rests
- 130kg Safe working load
- Range of accessories



Aston Heavy Duty Shower Commode

Key features

- Seat width 56cm (22")
- Seat depth 43cm (17")
- Seat height 51 – 56cm (20" – 22")
- Tool-free height and angle adjustable footplates
- Simple e-clip height adjustment to suit different toilet heights
- Armrests flip up for side transfers
- Rust proof aluminium frame
- Maximum Weight Capacity 260kg (40st)



Smit Care

SmitCare Apex Mobile Commode (SAC360)

Key features

- Four braked castors,
- Removable arms and swing-away, lift off footrests.
- Seat height 500mm (19.5")
- Seat size 435 x 460mm (17 x 18")
- Castors 130mm (5") diameter.
- 30 stone (190kg) max user weight limit

*NB. The products are not stainless steel and may rust. Geberit recommend use only on a short term basis / emergency use



Tips

Always specify that the shower chair is for use with the **Mera CARE** shower toilet at time of ordering

AquaMaster Range install toilet at 430mm seat height

All standard Freeway “Auto” Chairs - install toilet @465mm seat height

Tilt In Space Shower Chairs

Tilt in Space shower chairs have gained in popularity. Offering safe & secure support for both toileting and showering.

Osprey Healthcare, Freeway (Prism Medical), Care and Independence, Chiltern Invadex all have a range of upright shower chairs that are compatible with the Geberit AquaClean Mera Care shower toilet.



981 TIS Adult Tilt in Space Chair

Key Features

- Easy to Tilt
- Adjustable Headrest
- Adjustable Footrest
- Supportive Mesh
- Optional Mesh Aperture
- Stainless Steel (Rust resistant)
- Adjustable Seat Height



Ocean Ergo VIP

Key Features

- Stainless steel frame
- -5° to 40° seat tilt angle
- 480mm Seat width
- Modular Design
- Tilt in space
- Height Adjustable - 510mm – 610mm




Etac

The Swift mobile TILT

Key Features

- Unique tilt action
- Adjustable backrest
- Height adjustable
- Head support
- Head Support
- Tool free adjustments

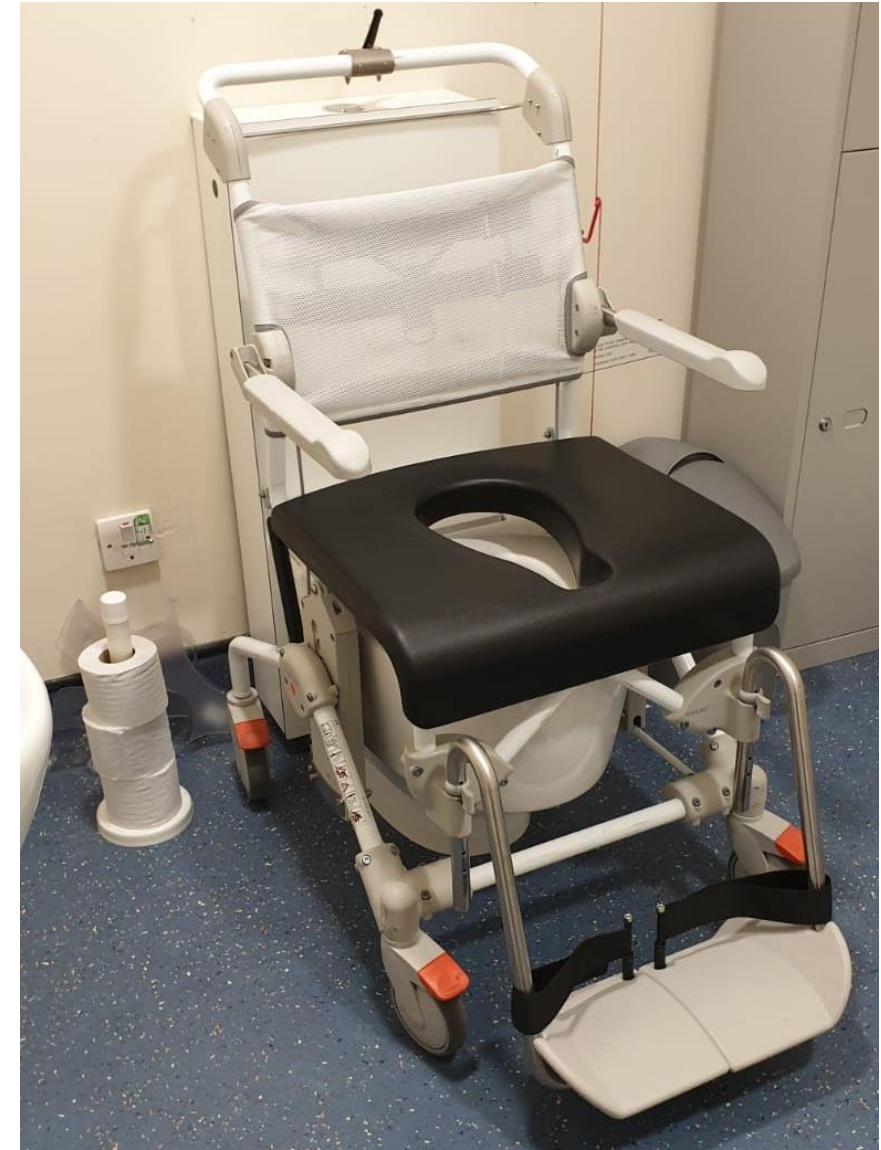

Freeway (Prism Medical)

T80 Geberit Mera CARE (Not AUTO)

Key Features

- Easy to Tilt
- Removable Headrest
- Adjustable Footrest
- Freecote protective frame covering
- Wide range of positioning accessories



Chiltern Invadex

Aqua Master Tilt in Space Shower Commode Chair

A16G – 18” seat width

A17G – 21” seat width

Key Features


- Full range of aperture, horseshoe and flat padded seats available
 - Option for bespoke seating, including pressure relief foam
 - Armrest, footrest and backrest options
 - Extensive range of accessories available
 - Dual gas rams to ensure smooth reliable operation
 - Corrosion resistant stainless steel and nylon braked castors
- Directional locking option available



The Raz AT Shower Chair

Key features

- Stainless steel chassis for durability
- Height adjustable seat
- Tension adjustable backrest - supports posture and provides lateral support
- Tilt-in-Space to 40° - excellent for pressure management and comfort
- Flip-up armrests for excellent access
- IPAS Seat System - adjust the seat depth to achieve optimum aperture positioning

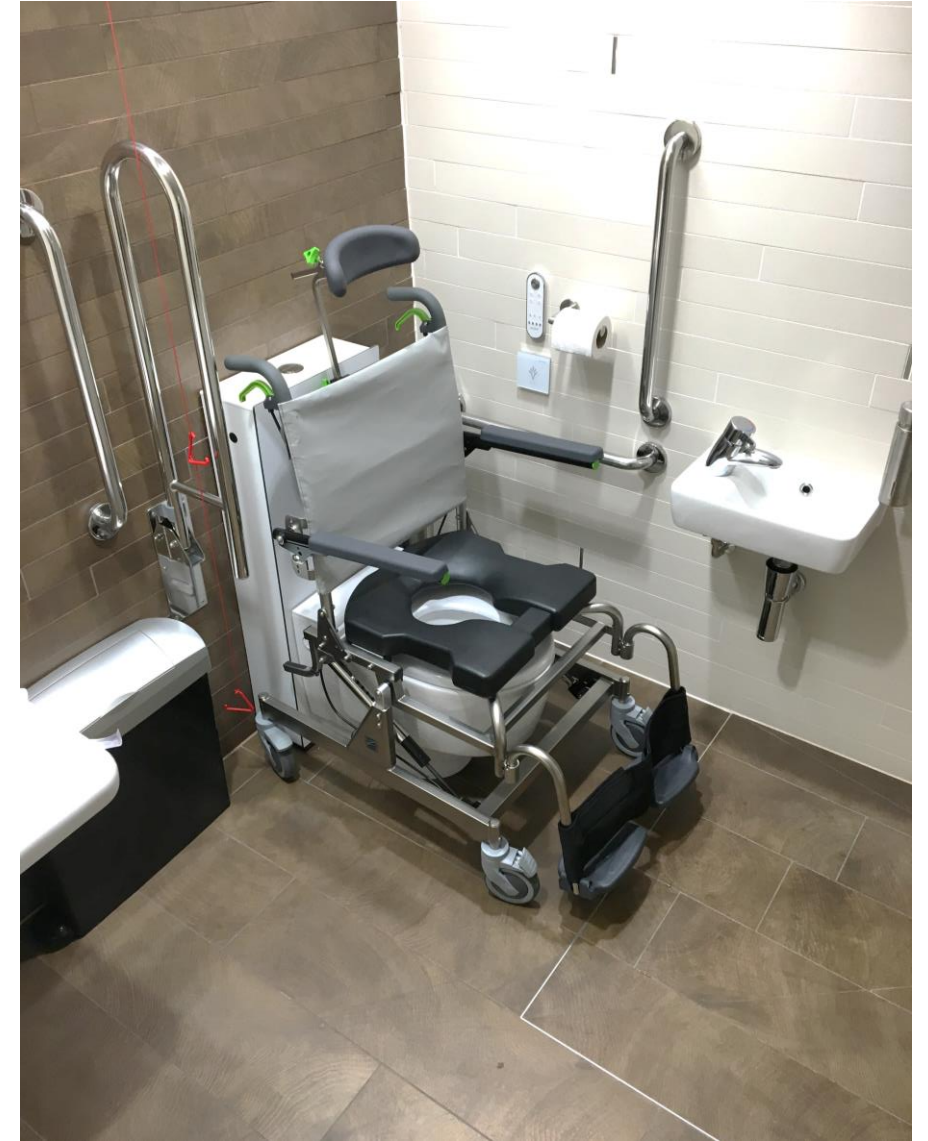

Tips

Freeway T80 Geberit
Mera Care - install
toilet at 465mm seat
height.

Always specify with Freeway to be
used with Mera CARE shower
toilet

AquaMaster
Tilt in Space
install toilet at
435mm seat
height

Osprey
Healthcare TIS
is height
adjustable to
match Geberit
seat heights

Always specify
Tilt in space
shower chair is
for use with a
shower toilet
when ordering

Paediatrics

Paediatric equipment has rapidly improved in recent years, offering greater support and helping to promote independence in a safe environment

Osprey Healthcare, Freeway (Prism Medical), Rifton and R82 are some of the market leaders in Paediatric shower chairs.

Other more basic equipment is also available via some high street stores and internet sellers.



Freeway (Prism Medical)

T90 Paediatric Self Propel Shower Chair

Key Features

- 365MM seat width
- Available as assistant or self propelled models
- Integral horizontal push handled backrest
- Pan sliders attach to the frame and are easily removable without tools
- Supplied as standard with straight paediatric armrests and small horseshoe seat
- Different height armrest sockets
- Different positions for armrest sockets
- Clearance on this frame is now 475mm – Install Geberit @ 465mm



Freeway (Prism Medical)

T80 Paediatric Reclining Shower Chair

Key Features

- Easy to Tilt
- Removable Headrest
- Adjustable Footrest
- Freecote protective frame covering
- Wide range of positioning accessories



Jiraffe (Rifton)

The HTS

Key Features

- Three seat sizes - all work with Mera **CARE**
- Tilt in Space
- Unique seat pad design
- Load bearing footplate
- Thoracic Supports
- Height Adjustable
- Seat depth Adjustable



R82

Flamingo

Key Features

- Four different sizes
- Back angle adjustment
- Height adjustable
- Tilt in Space
- Adjustable footplates
- Side Supports
- Height Adjustable
- Head Support



Argos / Amazon / Online

Children's toilet seat with step

Key Features

- Quick and affordable
- Overnight delivery
- Ideal for young children and individuals with Dwarfism
- Available in a wide range of colours



Tips

Rifton HTS & R82
Flamingo work best
with seat up –
contact Geberit
area manager at
time of install for
guidance

Children's
toilet seat with
step – Install
Geberit at
430mm

Install Geberit
at 465mm for
FreewayT80
Paediatric

All sizes of the
Rifton HTS work
over a Mera
CARE

The Smallest
Flamingo does
not work over a
Mera CARE

Bariatrics

With an increase in awareness of the needs of larger people and the adaptations needed for existing equipment to support a much larger body weight there has never been such a greater focus. Equipment is now available to suit all Plus size, Mediatrix and Bariatric users.

Equipment is now on the market capable of supporting users up to 85 stone



The Java Bench

Key Features

- Choice of Toilet or Horseshoe seat
- Skirt Included for Geberit compatibility
- Safe working load of 85 stone
- Slight incline on the seat to aid with standing & comfort
- Also compatible with other wash/ dry and standard toilets
- 19" or 21" seat height
- Choice of 2 leg styles, bolt down or rubber ferrules.
- 15 Year frame warranty



Gordon Ellis Seat & Frame Horse Shoe

Key Features

- Comfortable moulded plastic toilet seat
- Seat easily clips on and off the frame for cleaning
- 30 Stone weight limit



Langhams

Gordon Ellis Seat & Frame Full Aperture

Key Features

- Comfortable moulded plastic toilet seat
- Seat easily clips on and off the frame for cleaning
- 30 Stone weight limit



Aidapt Sussex Bariatric Commode

Key Features

- Vinyl padded over seat supplied for discreetness when not in use
- 7.5 litre commode bucket and lid provided as standard
- The Sussex commode ideal for users with limited mobility
- Can be used as a standalone commode or can be positioned over most WC suites
- Height adjustable legs and support arms as standard
- Maximum user weight limit 254kg (40 stones)



Tips

Removal of Seat gives better clearance for the “Shelf” and allows for full functional use of the wash and dry functions

Osprey seat height of 19” requires a Geberit seat height of 445mm.

Langham Multi frame also available with a big john seat attachment

Removal of toilet seat requires a Free of charge visit from a Geberit area manager

Toilet Seat Raisers

Toilet seat raisers have been in circulation for many years and have become more popular in recent times due to an increase in manufacturers reducing cost.

Based on the principle actions of a riser recliner chair a seat raiser can help the user to go from sit to stand and vice versa with ease. For those with more complex needs a verticle lift option is also available.



Drive Devilbiss Solo Toilet Lift

Key Features

- Battery powered device to get on and off toilet
- Helps reduce manual handling risks for carers
- Powered lifting replicates body's natural movement
- Seat can be stopped at any position to suit user
- Set of extended feet is available as an accessory to raise seat a further 6cm (2.5") for accommodating tall users or high toilets



Aerolet (KingKraft)

Aerolet sit to stand

Key Features

- Max user weight 180Kg
- Simple to use controls
- Incorporates Geberit own toilet seat for zero change to function
- Seat seal under frame ensure optimum drying



Aerolet (KingKraft)

Aerolet Vertical lifter

Key Features

- Max user weight 180Kg
- Simple to use controls
- Incorporates Geberit own toilet seat for zero change to function
- Seat seal under frame ensure optimum drying



Lift seat

Lift seat Home

Key Features

- Installs Anywhere
- Free-standing unit fits over any toilet
- No assembly/tools required – simply unpack
- Comes with AC plug and rechargeable battery pack
- Easy to Use
- Steel frame for stable support
- Simple, two-button controller
- Raises and lowers slowly and smoothly



SmartHome toilet raiser

SmartHome HS 130K

Key Features

- Incorporate a Geberit Mera Classic Wall hung toilet
- Option to tilt at the top of the lift to allow user easy exit from the toilet
- Discreet and modern looking
- Easy to use remote control



Other equipment

There are many basic products that fit around the toilet offering various levels of support or assistance. Often non specialist equipment can be as suitable

Drop down, wall mounted arms, toilet frame surrounds, MD frames are some of the most common accessories.



Gel Ovations

The Gel Seat

Key Features

- Offers Pressure relief whilst toileting.
- Quick and easy to remove
- Covered in waterproof Dartex material
- Non slip underside for safe transferring.



Toilet surround frames

Frames

Key Features


- Quick and easy to place
- Various manufacturers
- Stock Loan store item
- Height adjustable



Wall mounted drop down rails

Key Features

- Quick and easy to place
- Various manufacturers
- Available with in built toilet roll holder
- Various style and coatings

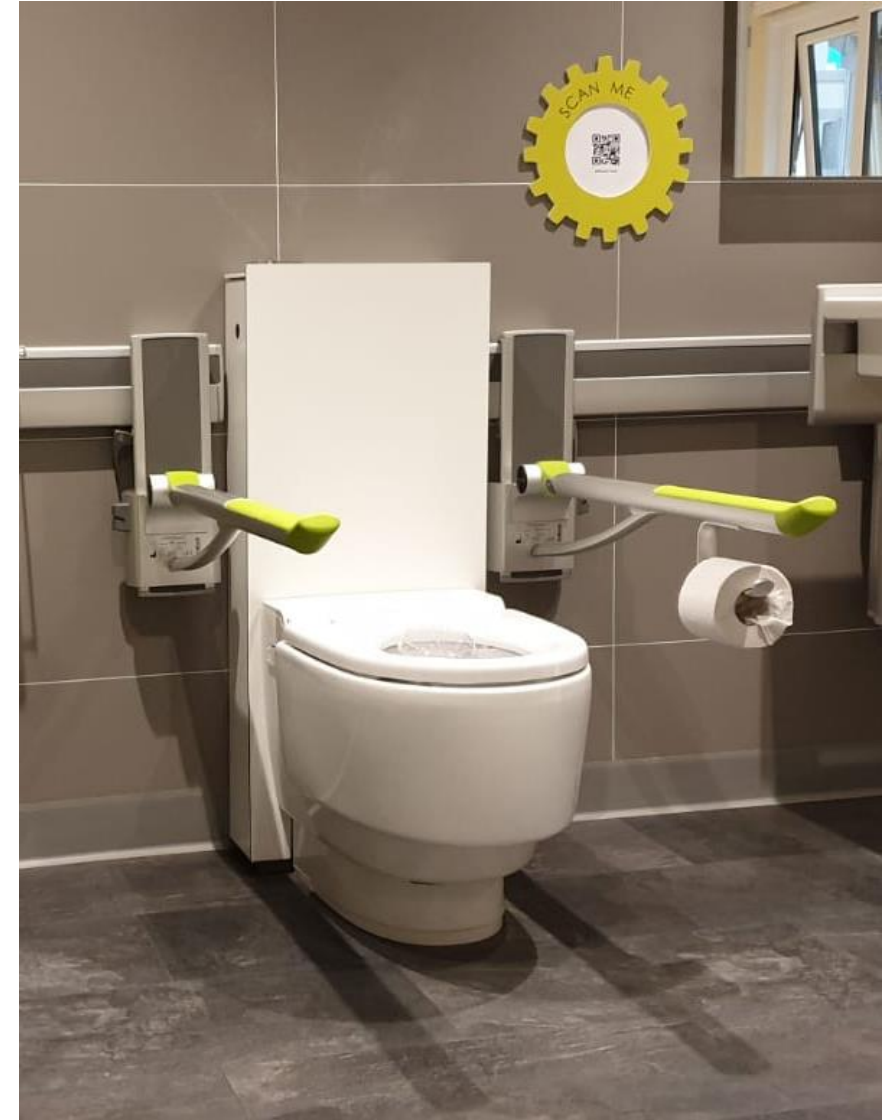

Easyfold Mobile Shower Commode

Key Features

- Aluminium Frame
- Blue PVC & Polyurethane Upholstery
- Removable Seat & Cover for Easy Cleaning
- Soft PU Non Slip Armrest
- 4 Locking Castors
- Removable Bucket (front removal)
- Retractable Footrest
- Shower cover for seat
- Foldable for Easy Storage and transport



Daily Care

The MD Frame

Key Features

- Installs Anywhere
- Free-standing unit fits over any toilet
- Screws to floor for stability
- Arm rest options
- Easy to Use
- Adjustable height arms and footplate



GEBERIT AQUACLEAN MERA CARE

**5 YEAR
WARRANTY**

FREE 5 years parts and labour
when you register your product
geberit-aquaclean.co.uk/care



■ **GEBERIT**

NEW

Standard Specification

- ✓ 5 year FREE Warranty
- ✓ 5 Profile settings
- ✓ Oscillating Wash
- ✓ Adjustable spray position
- ✓ Constant warm water
- ✓ Opto remote
- ✓ User remote control
- ✓ App enabled
- ✓ Lady wash
- ✓ Height adjustable
- ✓ Self servicing function

Optional Extras

- ✓ Toilet seat Lid (Product codes 243.211.11.1 & 243.404.00.1)



Manufacturer links

www.opsreyhealthcare.com

www.prismmedical.co.uk

www.chilterninvadex.co.uk

www.jiraffe.org.uk

www.r82.com

www.careandindependence.com

www.kingkraft.co.uk

www.dailycare.co.uk

www.langham-ge.co.uk

www.smarthomebathrooms.co.uk

www.invacare.co.uk

www.gelovationseurope.com




HOW TO SELECT ACCESSIBILITY (SHOWER CHAIR) MODE


Rear of remote control-LCD Panel and Cursors

1. Battery Compartment 2. LCD screen

3. Navigation/Selection Cursors

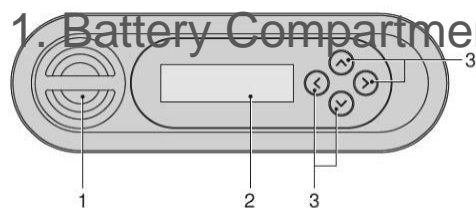


Step 1

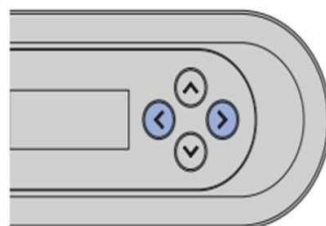

Press Left/right cursor to launch screen,
then scroll left/right until you see :

Care and Maintenance


Step 2

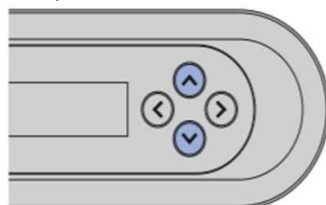

Scroll down until you see:

Accessibility Mode
< off on >


Step 3

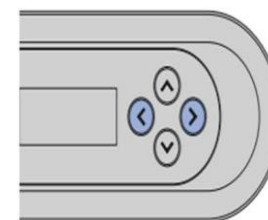

Scroll Left/Right until **on** is
underlined:


Accessibility Mode
< off on >

Scroll down until you see:

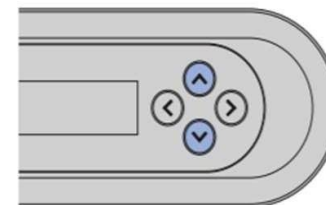

Activate Mode?
< off on >

Step 4

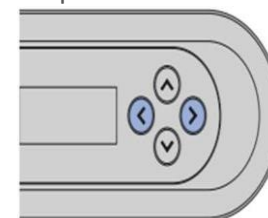

Scroll Left/Right until **on** is underlined:

Activate Mode?
< off on >

Scroll down to
save